

EZee Life™

Aluminum Wheelchairs



CH1096 – 18” Seat Width

CH1097 – 20” Seat Width

EZee Life™

15-34 Futurity Gate, Concord Ontario L4K 1S6

Specifications:

Seat Width:	18" (CH1096) 20" (CH1097)
Seat Depth:	16"
Seat Height:	20"
Frame type:	Folding Aluminum - Double Cross Brace
Arm style:	Desk Length Flip Back
Footrest Type:	60° Swing-away, detachable (17" - 21")
Foot Plates:	Flip up PVC
Rear Tires:	24" x 1" Flat free
Casters:	8" x 2" Flat Free
Wheel Locks:	Push to lock extended
Anti-Tippers:	Yes (4" Adjustment)
Weight Capacity:	300 lbs.
Weight:	36 lbs. (CH1096) 37 lbs. (CH1097)

Wheelchair Components



A. Push handles
B. Back upholstery
C. Rear wheel
D. Pushrims
E. Antitipper

F. Caster
H. Armrest
I. Armrest lever
J. Brake handle
K. Footrest
L. Footplate

Instructions

Removing the Armrests

Release the armrest by pushing in the push pin button (1) and pull the arm straight up using two hands (2)

REMEMBER removing the armrests can assist with side transfer on/off the bed, in/out of the car for example.

CAUTION when putting the armrests back on make sure they are locked in place before transporting.

Pull Upwards to Remove Arms (2)



Flip Back Arms (3)

Arm Pin Release (1)

Flip Armrests Back

To flip the arm rest back release the armrest push pin and lift the front of the armrest (3) upward.

Folding and Opening

- To fold, flip up the footplates or remove the footrests altogether and pull up on the center front & back of seat upholstery.
- To unfold, push down on the seat side frame until the seat locks into place. Put the footrests on and/or fold down the footplates.



Adjusting the Footrest Length

Using the wrench provided, place it over the bolt and turn to left to loosen or to the right to tighten in place. Adjust the length by sliding the footrest up or down so that the person's full thigh is supported by the seat.

REMEMBER Footplates need to keep at least a 2" clearance from the ground to overcome thresholds.

REMEMBER if the knees are too high the person may get a sore bottom from increased pressure. If the knees are too low the person may slide out of the chair



Removing & Replacing Footrests

Swing the footrest to the side by pushing the lever (1) Grasp the top bar of the footrest and lift up (2), releasing the footrests from the pins. Repeat on the other side.

Holding the footrest in one hand, from the side align the pins with the holes in the footrest and attach (3). Swing footrest back in place.



Applying the Brakes

Push the brake handle forward to lock and pull back to release. **REMEMBER** to apply the brakes before getting in or out of the chair.

Wheels and Casters

The wheels and casters have maintenance free solid tires installed that will not become flat if they are cut or punctured by debris on the ground.

The chair is fitted with 8” diameter casters and 24” diameter rear wheels to make travel over rough terrain and small obstacles easier.

On very rough or soft terrain it will be easier, and safer, for the care giver to pull the chair backwards until smoother or harder terrain is reached.

Safety Precautions

1. Consult your doctor, therapist, or other qualified professional to determine if this mobility device meets your specific needs.
2. Before using, make sure all parts are secure and the frame is fully opened
3. Ensure wheel locks are engaged before transferring to or from the wheelchair.
4. Use the supplied seat belt when being pushed by others.
5. DO NOT overload this wheelchair. The weight capacity is 300 pounds including the user and any things they may be carrying.

Inspection Items

Ensure the chair rolls straight (no excessive drag or pull to one side).

Inspect the upholstery for tears and/or sagging.

Inspect hand grips for wear, looseness or damage.

Inspect for any loose or broken hardware or parts.

Inspect for cracked, broken or loose spokes.

Ensure the hand brakes DO NOT interfere with the tires when rolling.

Ensure the hand brakes are easy to engage.





Limited Warranty

Ezee Life products carry a one year warranty against manufacturing defects, faulty materials and workmanship only. Any alterations, misuse, abuse, or accidental damage voids this warranty.

Factory Direct Medical reserves the right to replace or repair any part that has become defective. The warranty does not extend to consequential costs resulting from the fault or defect of the product: freight and travel costs, loss of earnings, or other expenses that one may incur.

Warranty is valid for the original purchaser only and the original receipt must be presented before any warranty options are considered. Please also be prepared to provide the date of purchase and serial number.

Purchase Information:

Date of Purchase: _____

Dealer Name: _____

Dealer Location: _____

Serial Number: _____

Imported / Distributed by:

Factory Direct Medical
34 Futurity Gate, Unit # 15
Concord, Ontario
Canada, L4K 1S6
416-739-8393
855-235-2400