EZee Life

Fauteuil roulant en acier

| | |
|---------|--------|
| # Item | # Ref |
| CRP0220 | CH1090 |

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| | |
|---------|--------|
| Item # | MFG # |
| CRP0220 | CH1090 |

Limited Warranty

EZee Life brand product have a Limited Warranty for one year from the date of purchase against faulty workmanship or materials.

In the event of a defect covered by this limited warranty, we will, at our option, repair or replace the device. Please contact your supplier/dealer should a fault occur.

The warranty does not extend to the consequential costs resulting from fault clearance, in particular freight and travel costs, loss of earnings, expenses, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

The manufacturer will not accept responsibility for any damage or injury caused by misuse or negligence or non-observance of the instructions set out above.

If you have a question about your **EZee Life** device or this warranty, please contact an authorized **EZee Life** dealer.

EZee Life

USE and SAFETY

Wheel Locks: Engage both wheel locks before getting in or out of chair, leaning forward in chair, or while in an elevator or wheelchair lift. Proper adjustment of wheel lock is at least 1/8” when locked. Locks are engaged by pushing handle completely forward. Adjustments to locks are made by loosening or tightening nut on the carriage bolt. Embed lock shoe at least 1/8” by sliding clamp toward rear wheel while handle is engaged in locked position. Tighten nut and bolt to secure in position. Test for correct locking action before actual use.

Tires: Solid rubber tires need minimal maintenance, but should be cleaned occasionally with a damp cloth. Replace if tires become severely worn or cracking appears.

Front Casters: Casters should be checked periodically and lubricated occasionally. Lubricate by removing fork and greasing caster stem bearings. Lubricate wheel bearings by removing wheel from fork. Reassemble fork and wheel by tightening nuts until no play remains, but wheels roll freely. If nut is too loose, caster will shimmy, if too tight, chair will be hard to steer.

Removal of Swing-away Foot rests/Leg rests:

Release lock by pushing on the front rigging lever. Swing-away foot rest /leg rest out and sideways. Lift to remove.

Adjusting Length of Footrest/Leg rest: When adjusting length of foot rests or leg rests, user should be sitting in wheelchair with feet on the foot plates. Loosen bolt on the clamp around support assembly. Raise or lower foot plate to desired position that will provide maximum

support of thighs. Front rigging should not be so low that it causes uncomfortable pressure to thighs at front of seat upholstery. Footrest should not be lower than 2-1/2” from the ground to ensure clearance.

Adjusting/Elevating Leg rest: Lift foot plate on leg rest assembly to desired position. Release leg rest assembly and it will automatically lock into place. To lower, push out on leg rest lever and lower foot plate to desired position.

Wheels: Large rear wheels should be inspected at least once a month to ensure proper tightness. If wheel wobbles or is loose, release lock nut and tighten axle bolt slightly. Test for proper tightness by learning chair to one side and spin adjusted wheel.

General Care: Wipe chrome parts with a clean, soft cloth at least once a week. All metal parts can be cleaned with auto wax or similar products. If exposed to moisture, dry immediately. Do not use an abrasive cleanser that will scratch the finish.

To Open Chair: Open wheelchair by tilting to one side and pushing down on seat rail until fully opened.

To Close/Fold Chair: First fold foot plates to vertical position. Grab seat with both hands at front and back edge and lift. Or, tilt chair to one side and lift up on seat rail.

Removable Arms: Remove arms by depressing spring button and lifting arms out of front and rear sockets.

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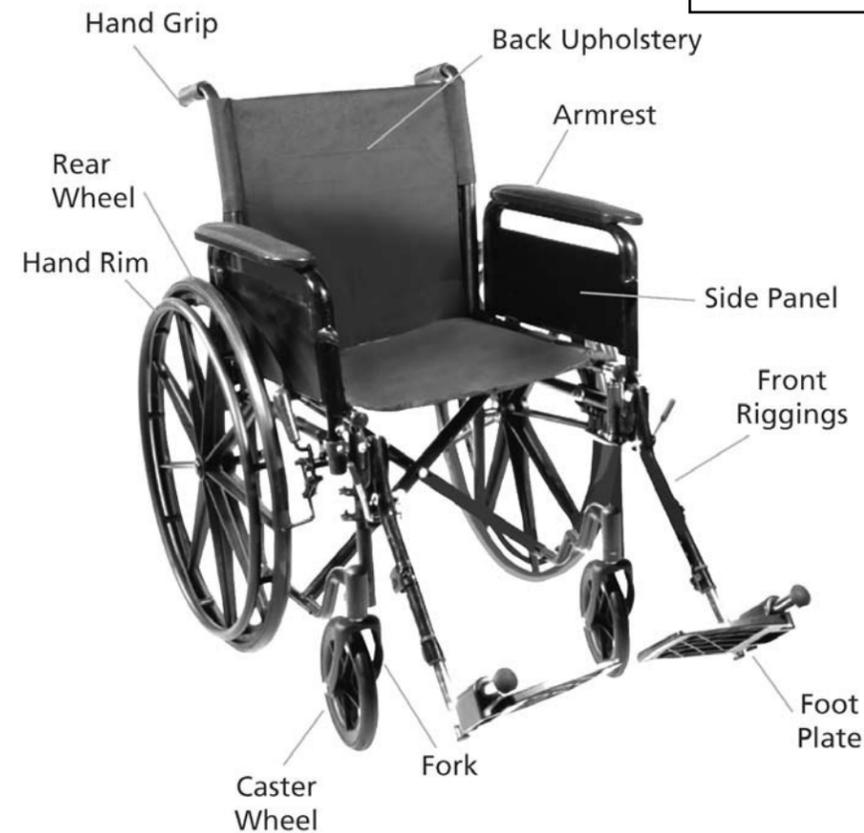
CAUTION:

Stability

- Wheelchair is designed to be safe and stable as long as center of gravity is maintained while going about normal activities.
- Care must be taken while performing activities that shift your center of gravity in the chair.
- Never reach so far that require you to shift in the seat or bend over out of the chair.
- When you have to reach for objects that require you to lean, be certain front casters are pointing in a forward position that extends the wheel base and will stabilize the chair.

Ramps

- Be sure of your own capabilities, limitations and strength before attempting to go up a ramp.
- Correct techniques on ramp negotiation should be shown to you by a qualified professional.
- Assistance is recommended when going up or down inclines greater than 10%.
- If you must stop on an incline, avoid sudden abrupt movement when you resume movement.
- Because of backward angle of the chair, sudden abrupt movement could cause instability or cause a sudden stop.
- Avoid changing direction when going down an incline, as this could result in instability of tipping over.

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